Aesculap Dental Implantology

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Dentistry is currently facing drastic changes. The current state of medical and technological knowledge means that operations can be performed with ever greater precision and using less and less invasive techniques. Along with the availability of increasingly sophisticated diagnostic tools, the decisive factor underlying this progress is the parallel ongoing development of the entire range of dental instruments.

With its new and innovative products, Aesculap is playing a decisive part in this development. Aesculap's products are developed strictly by the close contact to our customers.

This applies as much to new and innovative areas such as microsurgical instruments for implantology, periodontics and endodontics, as to standard dental instruments in general. Throughout the world, Aesculap products have proved themselves in day-to-day practice, not least from the point of view of quality and cost. With limited investment budgets available,

optimized hospital procedures and resulting efficiency gains offer a means of making better use of existing resources. Aesculap's range of services can help to achieve such improvements.

The following service components enable us to meet the ever increasing demands of the future:

- our many years of experience and competence in the field of planning and advice,
- the world renowned excellent quality of our products,
- our high level of product availability,
- individual and practice-oriented service concepts to maintain the value of your dental instruments,
- continuous new opportunities for ongoing and further training in the form of the courses offered by the Aesculap Academy.

With ERGOPLANT, the ergonomically designed instrument for dental implantology, Aesculap is setting new standards:

For the first time a complete and unitary product-line, which covers all indications, is available to the implantology focused dentist. Where ever necessary, the working tips had been optimized for each indication. Due to the ergonomic handle design the user is supported by easiness and grip. Finally the surgeon can concentrate on the patient respectively the surgery. The instrument handling is done intuitively.

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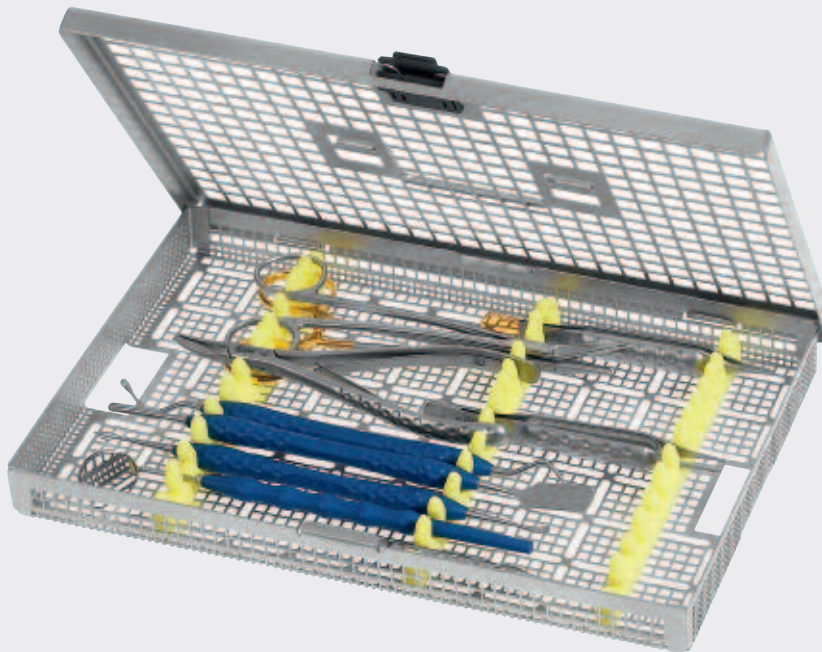
In close collaboration with **Dr. Dr. Andres Stricker of Freiburg University**, an instrument set especially designed for dental implantology has been developed for optimal ergonomics concerning the working tips and handle for any specific indication.

ERGOPLANT thus enables working efficiently with minimum physical strain under practice conditions.

All ERGOPLANT instruments are suitable for steam sterilization 134°C (Autoclav).

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Implantology Basic Kit

The standard instrumentation for all dento-alveolar operations.

DX705

Aesculap Basic Kit for dental implantology, consisting of:

BB515	1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
BB063R	1 Scalpel handle, No. 3
DX094	1 ERGOPLANT Mouth mirror holder with PEEK handle
DA036R	1 PAK = Package of 12 Rhodium-Mouth mirror, 22 mm diam.
DX051R	1 ERGOPLANT Tissue forceps
DX052R	1 ERGOPLANT Dissecting forceps
DX201R	1 ERGOPLANT Raspatory, BUSER
DX200R	1 ERGOPLANT Raspatory, PRICHARD
DX220R	1 ERGOPLANT Bone curette, LUCAS
BM065R	1 DUROGRIP Needle holder, HEGAR-MAYO
DO219R	1 Gum scissors, LOCKLIN
DX500R	1 ERGOPLANT Bone rongeur, LUER-FRIEDMANN
JG381R	1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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Implantology Advanced Kit
Special instrumentation designed for the particular requirements of the bony implant site.

With this instrumentation it is possible to remove inflamed soft tissue residue completely while achieving maximum preservation of alveolar bone.

This reduces the need for augmentative measures.

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DX706

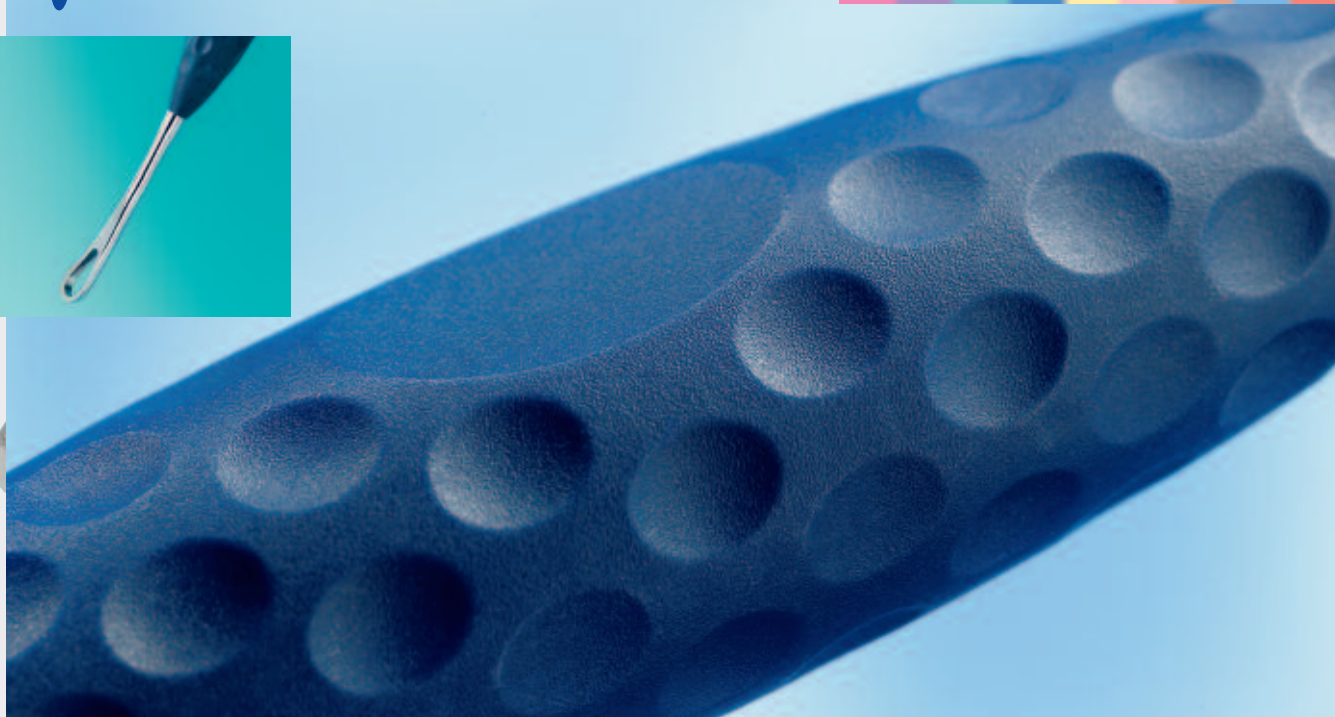
Ideal for detailed preparation of the boney implant site and for thorough removal of soft tissue residue. Also suitable for minor augmentative procedures.

Aesculap Advanced Kit for dental implantology, consisting of:

DX230R 1 ERGOPLANT Excavator
DX231R 1 ERGOPLANT Excavator
DX241R 1 ERGOPLANT Universal curette, M23A
DX300R 1 ERGOPLANT Furcation probe, NABERS
DX400R 1 ERGOPLANT Periotome
DX253R 1 ERGOPLANT Bone file, MILLER
DX252R 1 ERGOPLANT Bone file, MILLER
DX555R 1 ERGOPLANT Applicator for bone material, STRICKER

DX600R 1 ERGOPLANT Membrane placing instrument, STRICKER
DX501R 1 ERGOPLANT Bone cutting forceps
JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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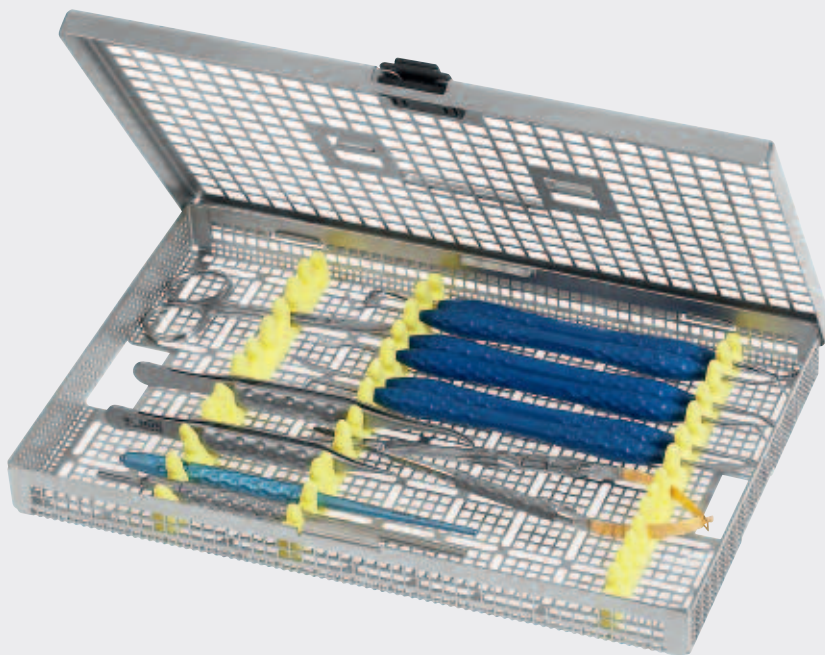
Filigree-aesthetic soft tissue management in implantological operations as the precondition for a good implantation.

These instruments can be used as successfully for larger or coarser mucocutaneous incisions as for the sensitive, easily torn mucous membrane in the area of the neighbouring teeth and their papillae. In soft tissue augmentations, such as free mucous membrane or connective tissue transplants, both harvest and transplantation site can be prepared simultaneously in exemplary fashion.

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DX707

Aesculap Soft Tissue Kit for dental implantology, consisting of:

BB045T	1 Titanium scalpel handle
BB367R	1 PAK = Package of 10 micro scalpel blades, sterile, Fig. 367
BB063R	1 Scalpel handle, No. 3
BB515	1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
DX050R	1 ERGOPLANT Tissue forceps
BM003R	1 DUROGRIP Needle holder
DX450R	1 ERGOPLANT Gingivectomy knife, KIRKLAND
DX203R	1 ERGOPLANT Raspatory
DX202R	1 ERGOPLANT Raspatory, STRICKER
DX405R	1 ERGOPLANT Atraumatic papilla elevator, PHW
DX401R	1 ERGOPLANT Spatula, sharp, STRICKER
DX402R	1 ERGOPLANT Spatula, HEIDEMANN
BC061R	1 Delicate scissors, curved
DX055R	1 ERGOPLANT Soft Tissue Forceps
JG381R	1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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In case of poor bone thickness up to 3mm, it's possible to split the jaw-bone (Bone Split) and separate it by means of special chisels, the so called Osteotomes. Finally a gap occurs between buccal and lingual respectively buccal and palatinal bone-lamella, where we can insert an implant.

Another indication could be the "chiseling out" of bone-transplants. In any case the user is supported by the ERGOPLANT handle design. Together with Dr. Stricker, we developed a special "Step-Osteotome". Due to this innovative instrument, cracks on the bone-lamella, as often occurs particularly on thin jaw-bones, can be avoided now.

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DX708

Aesculap Bone Split Kit for dental implantology, consisting of:

DX510R	1 ERGOPLANT Osteotome, straight, 3 mm
DX511R	1 ERGOPLANT Osteotome, straight, 4 mm
DX512R	1 ERGOPLANT Osteotome, straight, 6 mm
DX513R	1 ERGOPLANT Osteotome, straight, 8 mm
DX517R	1 ERGOPLANT Osteotome, curved, 4 mm
DX518R	1 ERGOPLANT Osteotome, curved, 6 mm
DX522R	1 ERGOPLANT Step osteotome, 6 mm
JG381R	1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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Under circumstances the achievement of primary stability in implantations could be problematically. For instance if the upper jaw's bone structure is soft and "comb-like". With the Bone Condenser Kit the bone can be condensed (Bone Condensing) step-by-step until we finally achieve enough primary stability. The aligned conical working tips are convex on the tip. The rather spongy bone is almost "transformed" into compact cortical bone, which now enables a secure and successful implant insertion.

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DX709

Aesculap Bone Condenser Kit for dental implantology, consisting of:

- DX530R 1 ERGOPLANT Bone condenser, straight, 1.5 – 2.2 mm
- DX531R 1 ERGOPLANT Bone condenser, straight, 2.2 – 2.8 mm
- DX532R 1 ERGOPLANT Bone condenser, straight, 2.8 – 3.5 mm
- DX533R 1 ERGOPLANT Bone condenser, straight, 3.5 – 4.2 mm
- DX534R 1 ERGOPLANT Bone condenser, straight, 4.2 – 4.8 mm
- DX535R 1 ERGOPLANT Bone condenser, bayonet, 1.5 – 2.2 mm
- DX536R 1 ERGOPLANT Bone condenser, bayonet, 2.2 – 2.8 mm
- DX537R 1 ERGOPLANT Bone condenser, bayonet, 2.8 – 3.5 mm
- DX538R 1 ERGOPLANT Bone condenser, bayonet, 3.5 – 4.2 mm
- DX539R 1 ERGOPLANT Bone condenser, bayonet, 4.2 – 4.8 mm
- JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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In case of upper-jaw augmentation, it is possible to pass the external sinus elevation with its lateral window technique, if still enough vertical bone is available. Then an internal Sinus Elevation by means of the Sinutome Kit can be proceeded. With different sizes of Sinutomes the maxillary sinus floor is elevated through the oral cavity towards cranial. The concave working tip is semi-sharp on the edge, which enables the taking along of additional bone for further elevation. After placing the augmentation-material, the implant can be inserted in the same session.

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DX710

Aesculap Sinutome Kit for dental implantology, consisting of:

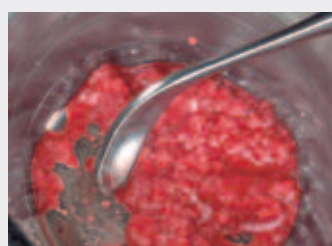
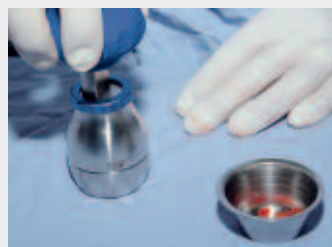
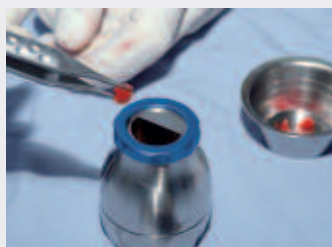
- DX570R 1 ERGOPLANT Sinutome, straight, Ø 2.2 mm
- DX571R 1 ERGOPLANT Sinutome, straight, Ø 2.8 mm
- DX572R 1 ERGOPLANT Sinutome, straight, Ø 3.5 mm
- DX573R 1 ERGOPLANT Sinutome, straight, Ø 4.2 mm
- DX574R 1 ERGOPLANT Sinutome, straight, Ø 4.8 mm
- DX575R 1 ERGOPLANT Sinutome, bayonet, Ø 2.2 mm
- DX576R 1 ERGOPLANT Sinutome, bayonet, Ø 2.8 mm
- DX577R 1 ERGOPLANT Sinutome, bayonet, Ø 3.5 mm
- DX578R 1 ERGOPLANT Sinutome, bayonet, Ø 4.2 mm
- DX579R 1 ERGOPLANT Sinutome, bayonet, Ø 4.8 mm
- JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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Autogenous bone in augmentation has been established as the "golden standard". The region where to gain the bone (Bone Harvesting) and the required consistence for augmentation can vary, depending on the case. Here the ERGOPLANT Micro Bone Mill shows its flexibility whereat the maximum yield of bone-grist is always ensured.

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DX800

Aesculap Micro Bone Mill Kit,
consisting of:

- | | |
|--------|--|
| DX801R | 1 ERGOPLANT Micro bone-mill
(incl. cutting disk, coarse) |
| DX802R | 1 ERGOPLANT Cutting disk, fine |
| DX810R | 1 ERGOPLANT Bone mill-cleaner, fine |
| DX811R | 1 ERGOPLANT Bone mill-cleaner, coarse |
| JF284R | 1 Siebtray (274 x 172 x 60 mm)
with storage aid for micro bone-mill,
suitable for dental container JN095 |
| TE888 | 1 Packing stencil (only available for DX800) |

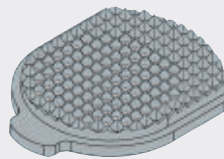
ERGOPLANT – Bone Harvesting

DX801R

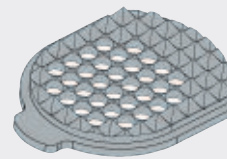
ERGOPLANT
Micro Macinino
(fornito con griglia a fori
grossi DX803R)

ERGOPLANT
Micro bone-mill
(delivered with coarse cutting
disk DX803R)

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ERGOPLANT Micro bone-mill

- small and handy
- interchangeable cutting disks
- cutting disks available for finer or coarser bone particles
- easy to clean due special cleaning instruments
- easy mounting and demounting
- maximum bone yield requires less bone harvesting

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Recommended accessories for the micro bone-mill:
Cleaning instruments DX810R and DX811R as shown on page 20.

Accessori raccomandati per il Micro Macinino ERGOPLANT Recommended Accessories for the ERGOPLANT Micro bone-mill

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Cleaning spatula for radial bone material collecting by cleaning the cutting disk's edge (DX811R the complete surface) at the same time. Valuable bone material can be used for augmentation.

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Enables secure carrying and placing of enough bone material. The other working tip serves for pushing and plugging bone material into the augmentation area.

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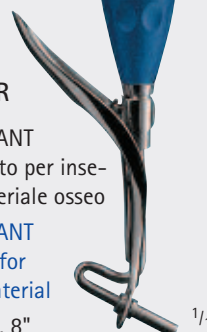
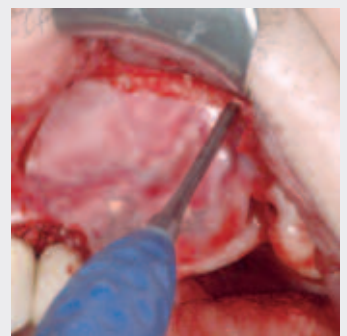
ERGOPLANT
Applicator for
bone material
195 mm, 7 3/4"

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The small surface of the pointed tip
avoids the sticking of the humid
membrane. The small spatula on the
other end, helps to take out bone
material of, for instance, the bone files
DX252R-DX253R, as shown on page 23.

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DX501R

ERGOPLANT
Pinza ossivora, curva

ERGOPLANT
Bone cutting forceps, curved

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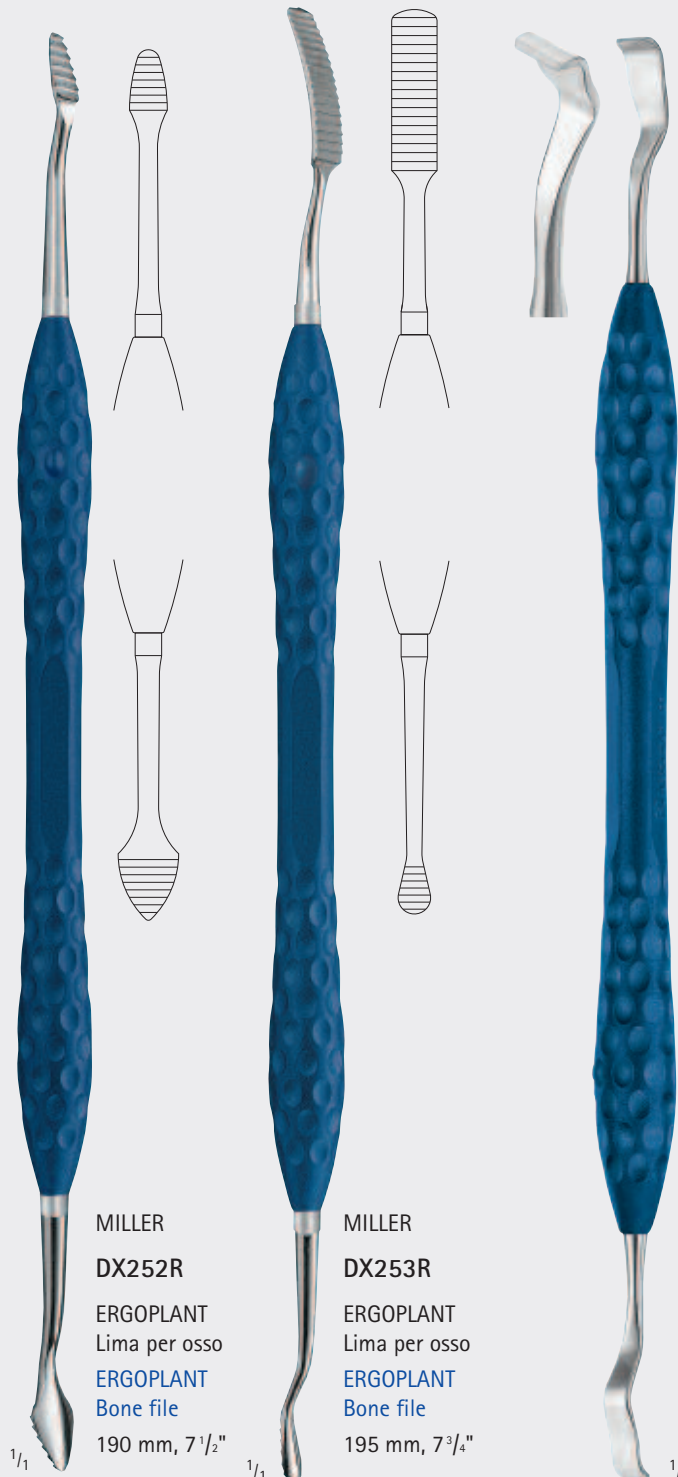
For detailed grabbing and removing of granulated tissue. Also suitable for cutting smaller bone transplants.

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If smaller quantity of bone material is required, with the bone files DX252R-DX253R it can be harvested directly from the jaw-bone, without additional trauma for the patient. The quantity is often sufficient, for augmentation around the implant-neck.

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With this raspatory fine bone particles can be harvested by simply scratching over the jaw-bone.

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The ERGOPLANT Immediate Implant Retractor ensures a safe implantation by holding back the alveolar soft tissue.

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Safe implantation without contamination!

In case of immediate implantation there is a great risk to contaminate the jaw-bone with connective tissue fibre of the soft tissue. The ERGOPLANT Immediate Implant Retractor ensures a safe implantation by holding back the alveolar soft tissue. Thereby working will be safer, easier and more comfortable.

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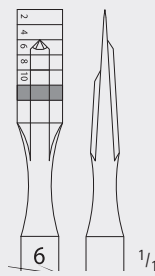
ERGOPLANT
Mallet 135 g
with interchangeable
plastic disks,
200 mm, 8"

$\frac{1}{1}$

STRICKER

DX522R

ERGOPLANT
Osteotomo a gradino
6 mm

ERGOPLANT
Step osteotome
6 mm

160 mm, 6 $\frac{1}{8}$ "

● Marcatura laser ogni 2 mm
laser marking in steps of 2 mm

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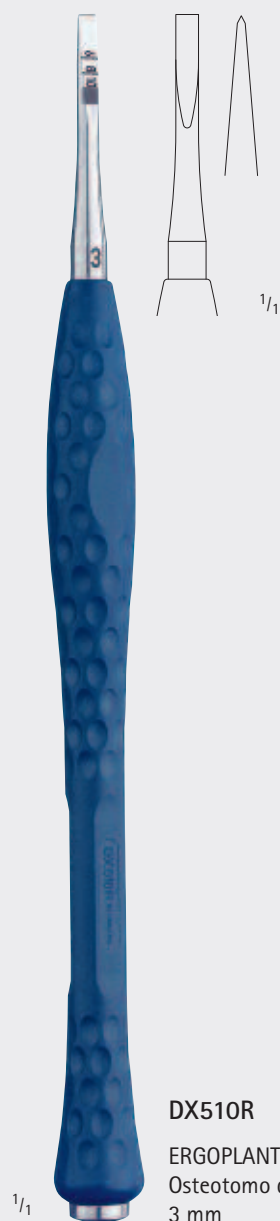
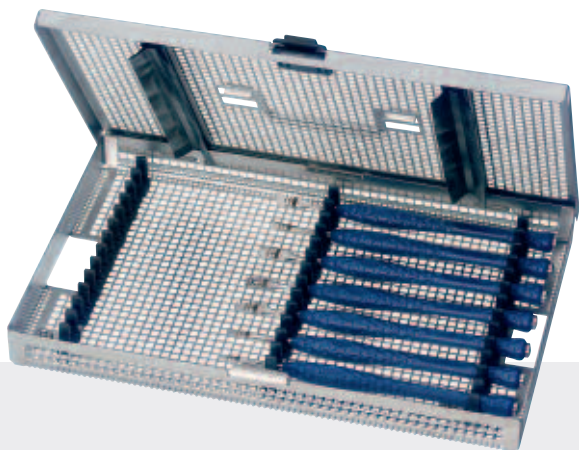
The Step-Osteotome was specially developed for secure bone-splitting with simultaneous stepwise widening. Due to the specially designed working tips, the maxillary crest will be first expanded towards palatal and only afterwards towards vestibular. In the time spongy bone in the center will be condensed. After removing the Step-Osteotome, a pre-selection point for the implant bur can be clearly seen.

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The maxillary crest is splitted transversal by the step osteotom.

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After the bone split is kept apart by the retractor.

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The implant can be positioned optimally by using the existing bone.

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After splitting the maxillary crest the bone split is closing within a short period of time. This leads to an unwished and unnecessary bone loss when drilling afterwards. This causes the risk of an implant that is not ideally positioned and additional augmentative treatments could be necessary.

The bone split is kept apart by the retractor. Thus, the existing bone can be used optimally to position the implant.

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DX505R

ERGOPLANT
Manico per cunei
ERGOPLANT
Applicator for
Bone split retractors

¹/₁

¹/₁

DX506T

ERGOPLANT
Cuneo in titanio
3 x 5 mm
ERGOPLANT
Titanium Bone split
retractor, 3 x 5 mm

¹/₁

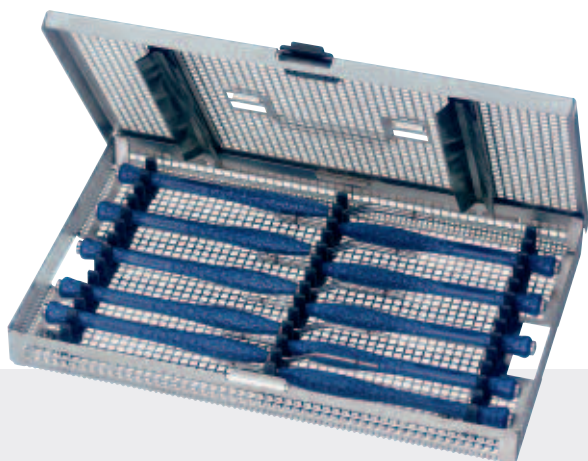
DX507T

ERGOPLANT
Cuneo in titanio
4 x 6 mm
ERGOPLANT
Titanium Bone split
retractor, 4 x 6 mm

DX509R

Rastrelliera per 4 cunei in titanio,
inseribile nei trays Aesculap.
Rack for 4 Titanium Bone split retractors,
can be easily fastened in Aesculap Dental
Siebtrays, incl. 2 fastening pins

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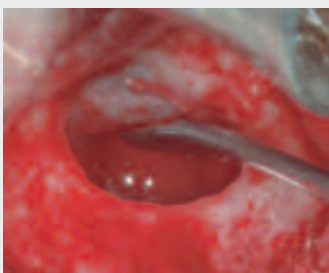
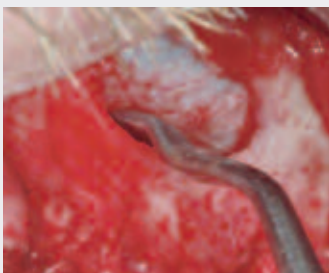
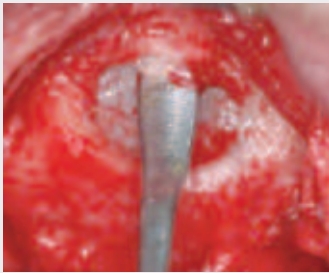
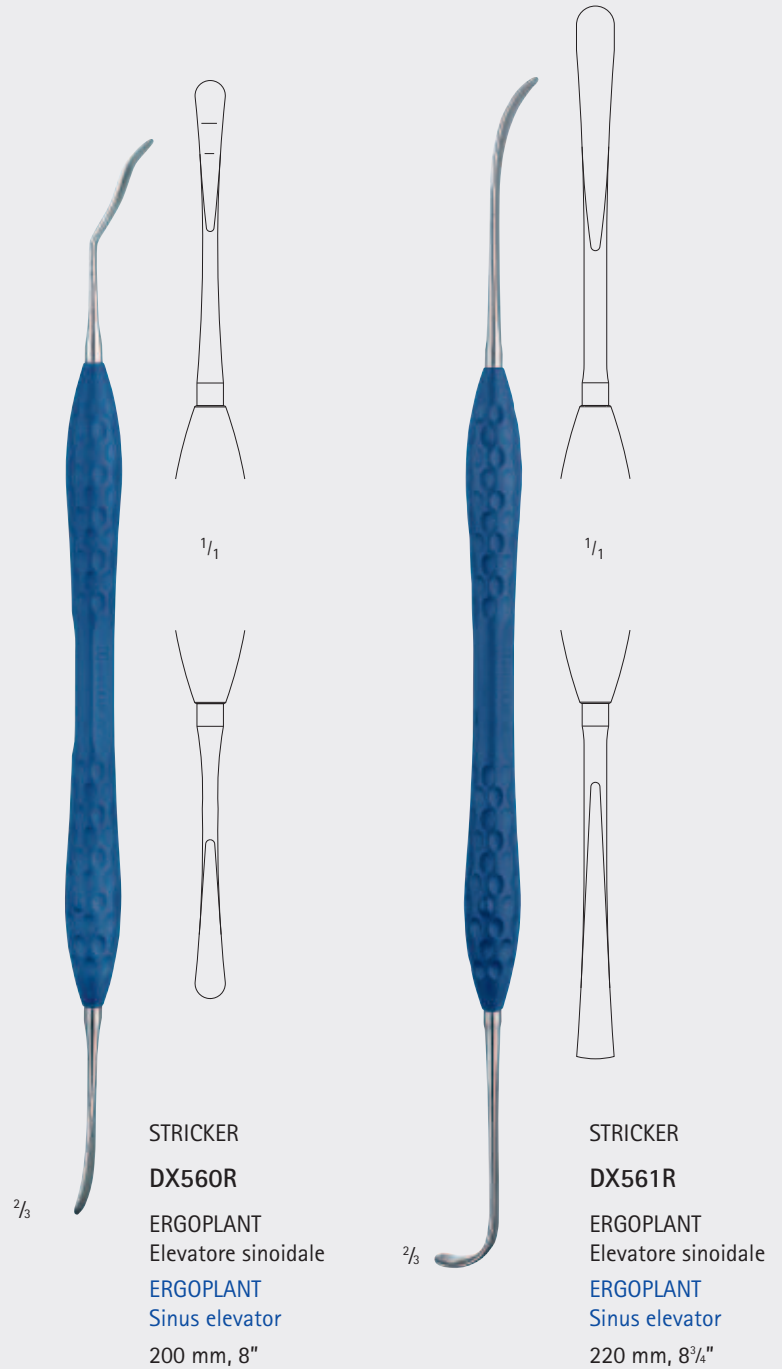
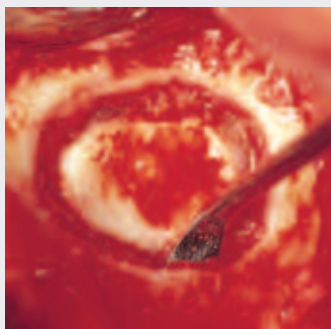
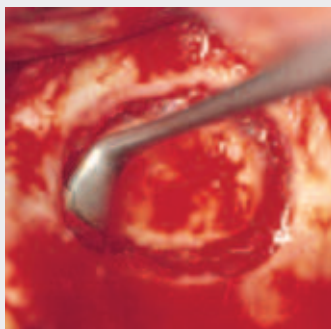
Separating the maxillary sinus membrane carefully from the bony base. Finally the sinus window will be pushed with the other working tip gently into the hollow maxillary cave.

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The Schneiderian membrane of the sinus can be detached with elevator D0571R by 1– 2 mm along the total cortical bone

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Elevator D0573R is particularly useful in the lower part for detaching the membrane further. The same goes for the caudal area, especially when the membrane is 4–5 mm above the base of the maxillary sinus and the maxillary sinus is built up with granular material.

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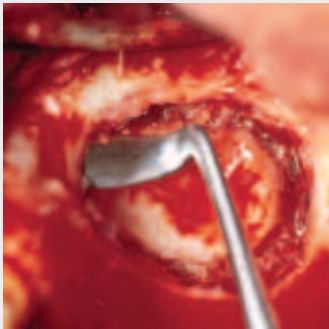
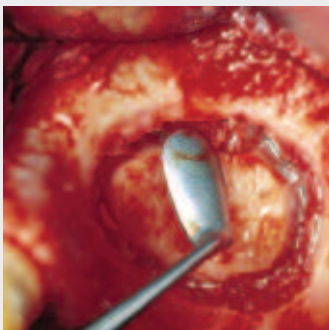
Elevator D0575R makes it possible to lift the membrane further in mesial region. One working tip is specially designed for the right maxillary sinus and the other for the left maxillary sinus.

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With elevator D0574R, the sinus membrane is further elevated in the upper and posterior part of the maxillary sinus wall.

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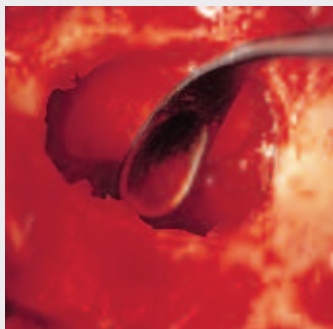
With the aid of elevator D0572R, the sinus membrane is further detached over a broad area in the inner region, especially along the roof of the palate. Condenser D0580R permits positioning of the filling material.

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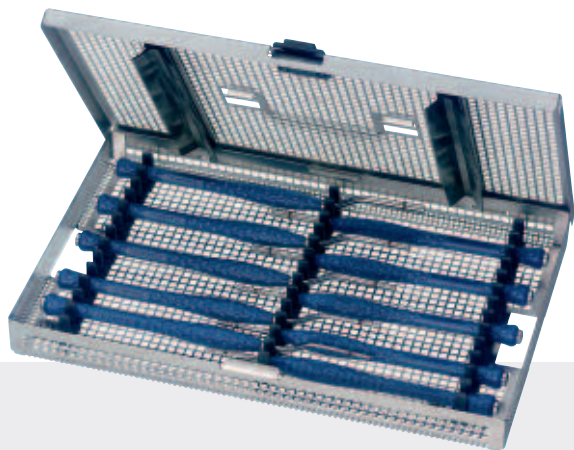
Reduced sinus lift instrument set.

Two double-ended sinus lift instruments consisting of D0576R (5 mm condenser/spoon elevator) and D0577R (90° dissector / raspatory).

Many years of experience with sinus lift instruments has shown that this reduced instrument set is more than sufficient for surgical procedures.

The reduced set appreciably lowers the investment the hospital or clinic needs to make.

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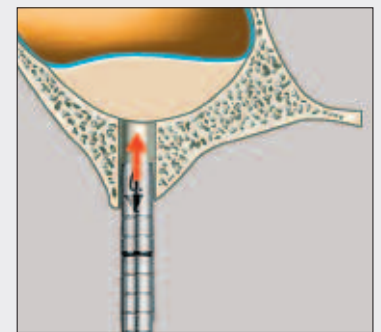
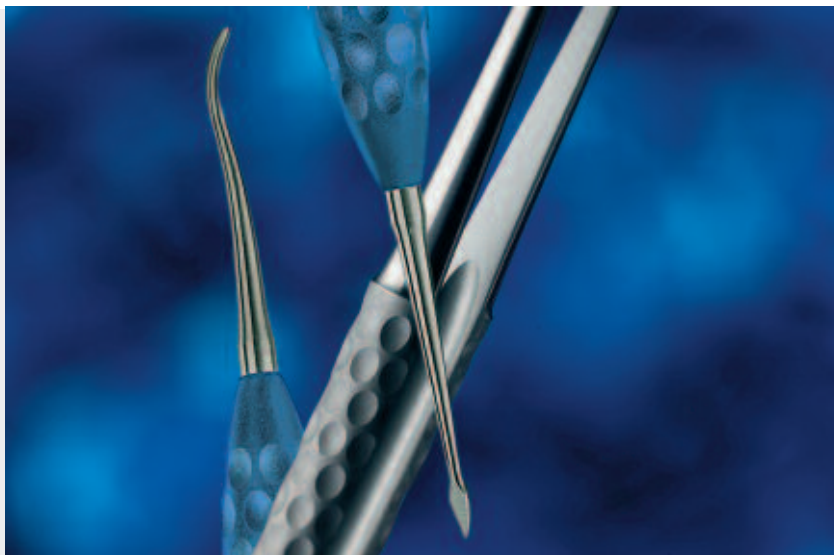
This illustration shows the technique
of the pericrestal sinus elevation.

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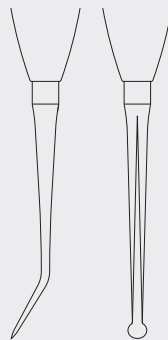
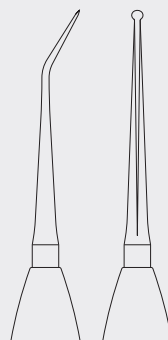
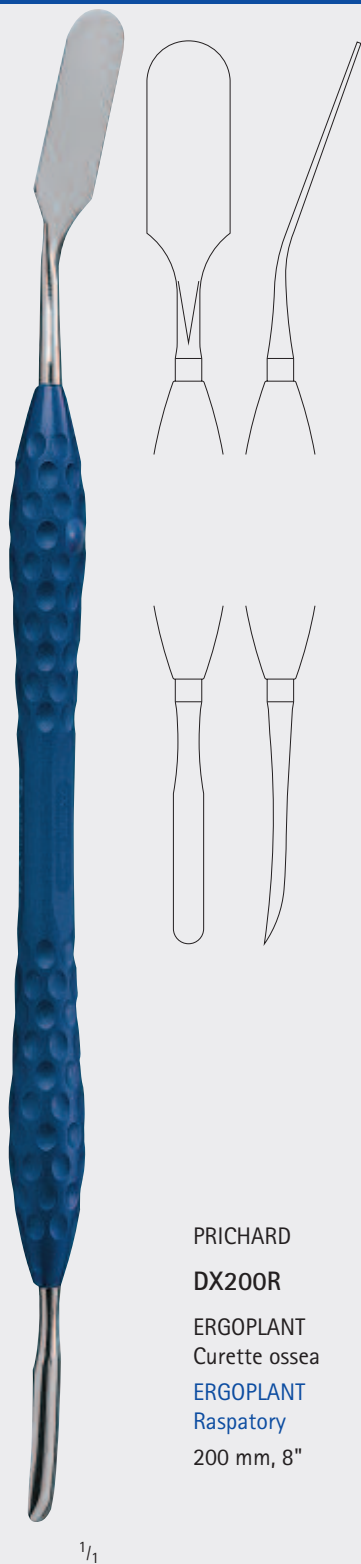
Atraumatic dissecting of the flap by means of the papilla elevator without any damage to the papillae.

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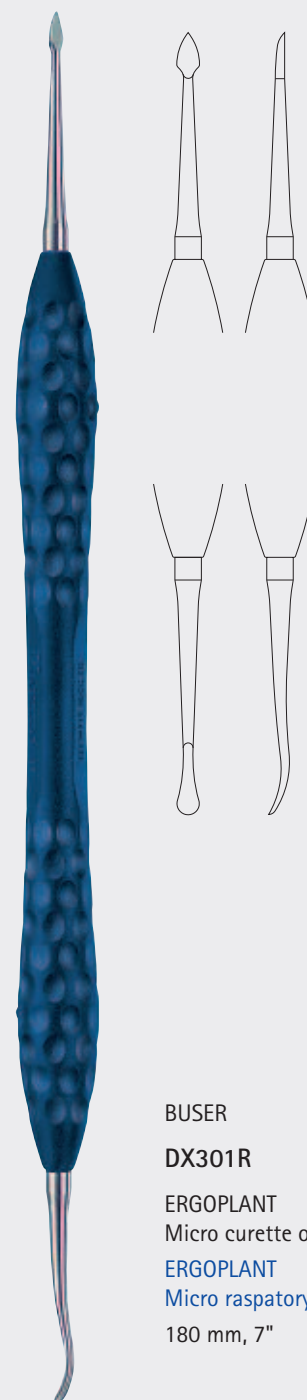
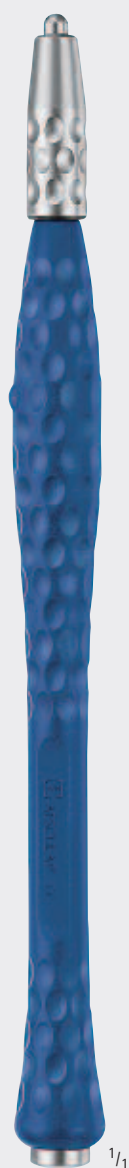
DX720

Micro Soft Tissue Set acc. Steigmann, consisting of:

- DX305R Handle for micro blades, 150 mm
- DX301R Micro raspatory, 180 mm
- DX303R Micro tissue forceps, 180 mm
- FD252R Micro ligature scissors, curved, one blade serrated, 180 mm
- FD253R Micro dissecting forceps, 180 mm
- BM003R DUROGRIP Needle holder, 170 mm
- JG381R Tray, 274 x 172 x 30 mm

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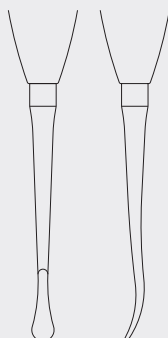
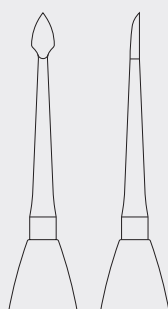
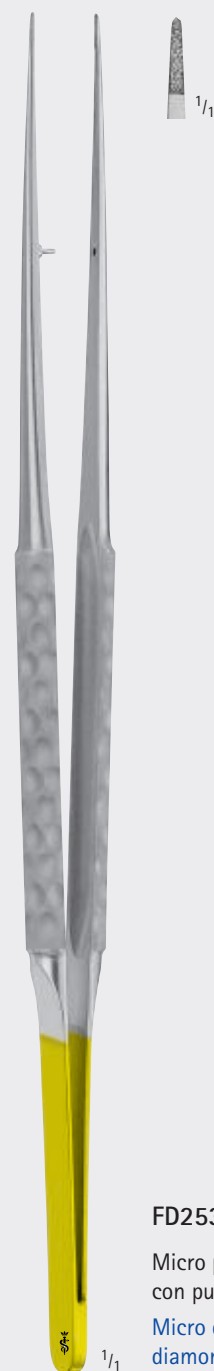
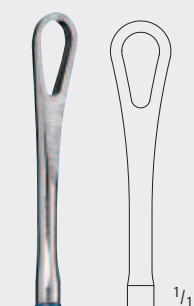
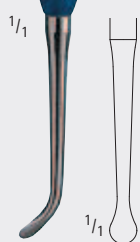
This working tip helps in case of suturing mucosa transplants, or as raspatory for larger surfaces.

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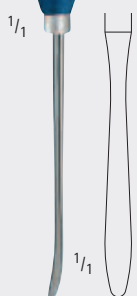
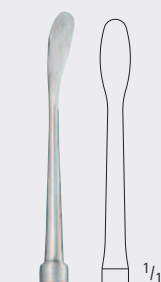
ERGOPLANT
Raspatory for fine soft tissue

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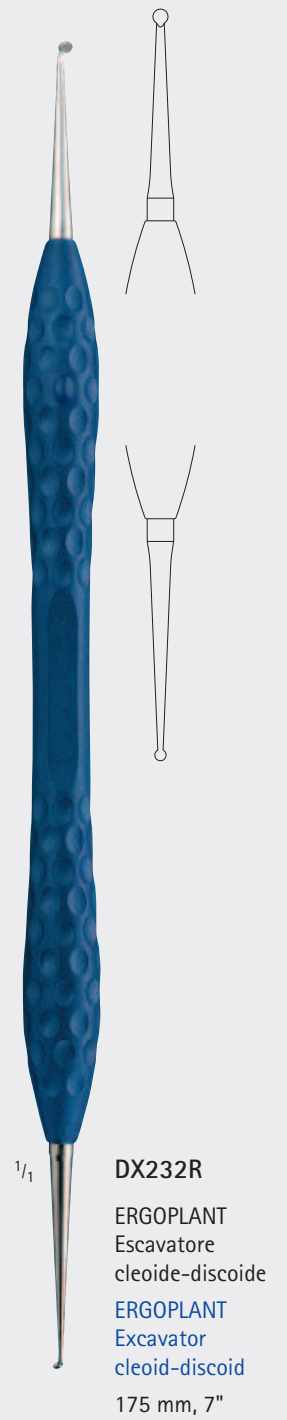
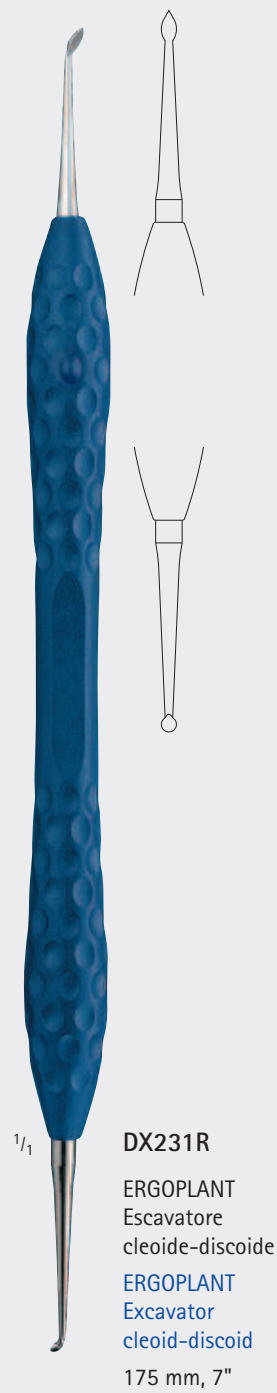
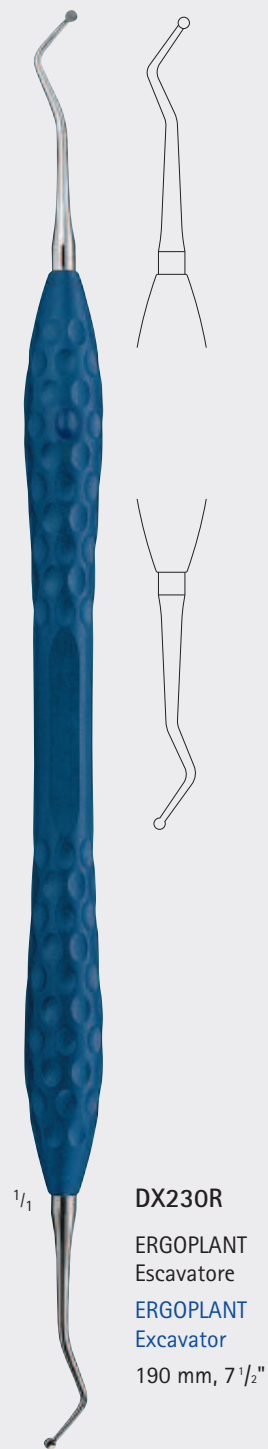
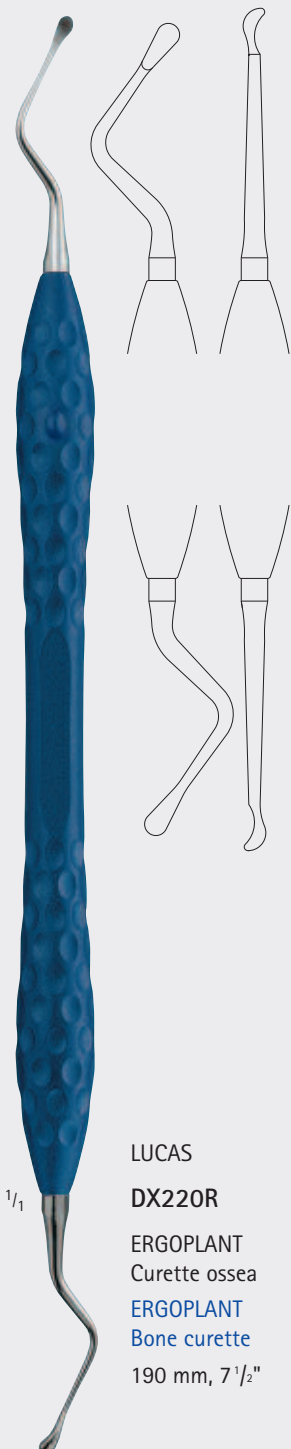
ERGOPLANT
Raspatory for soft tissue sharp/blunt

195 mm, 7 3/4"

McCALL
DX240R
M17S/18S
ERGOPLANT
Curette per tessuti infetti
ERGOPLANT
Curette for infected tissue
175 mm, 7"

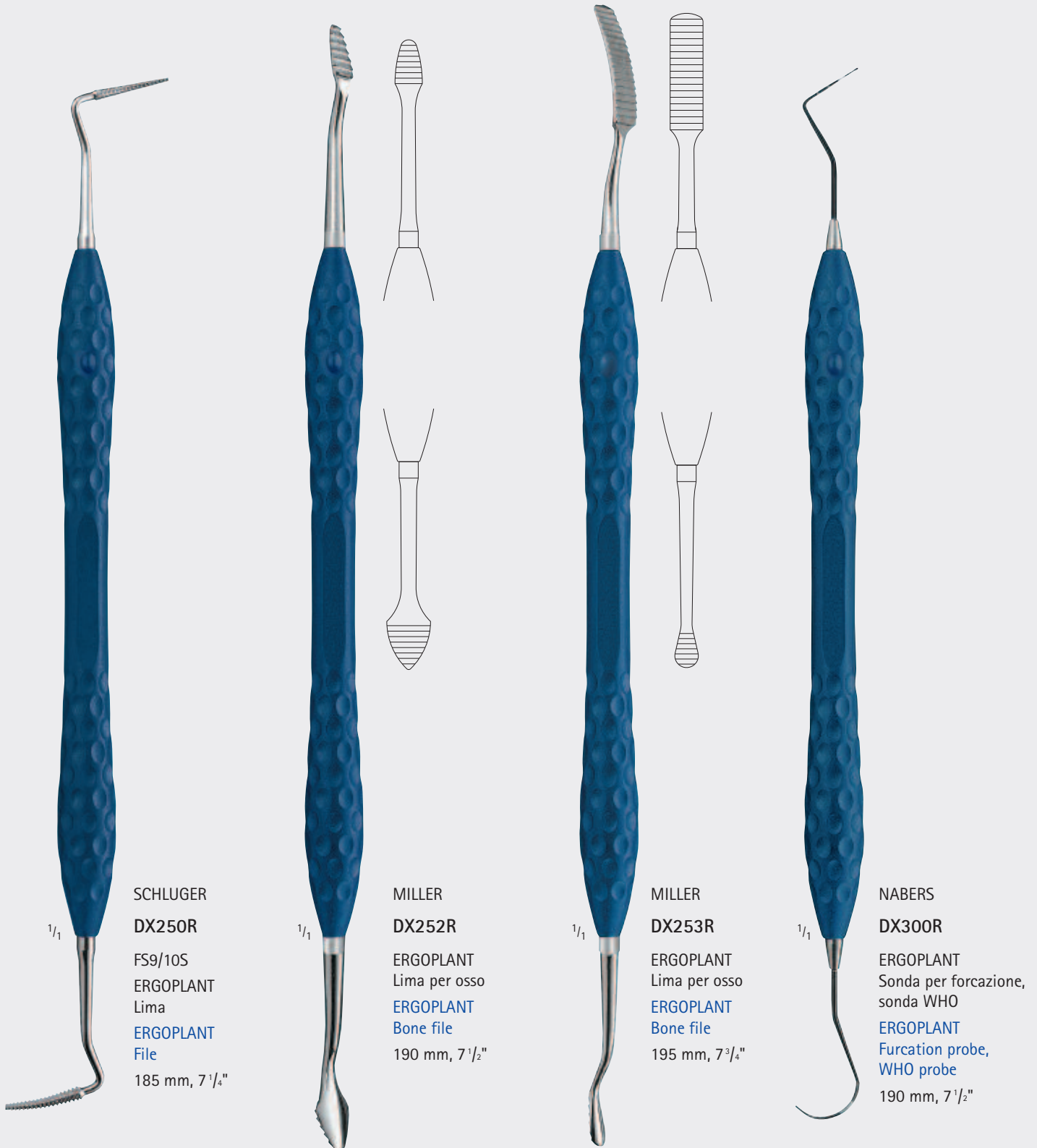
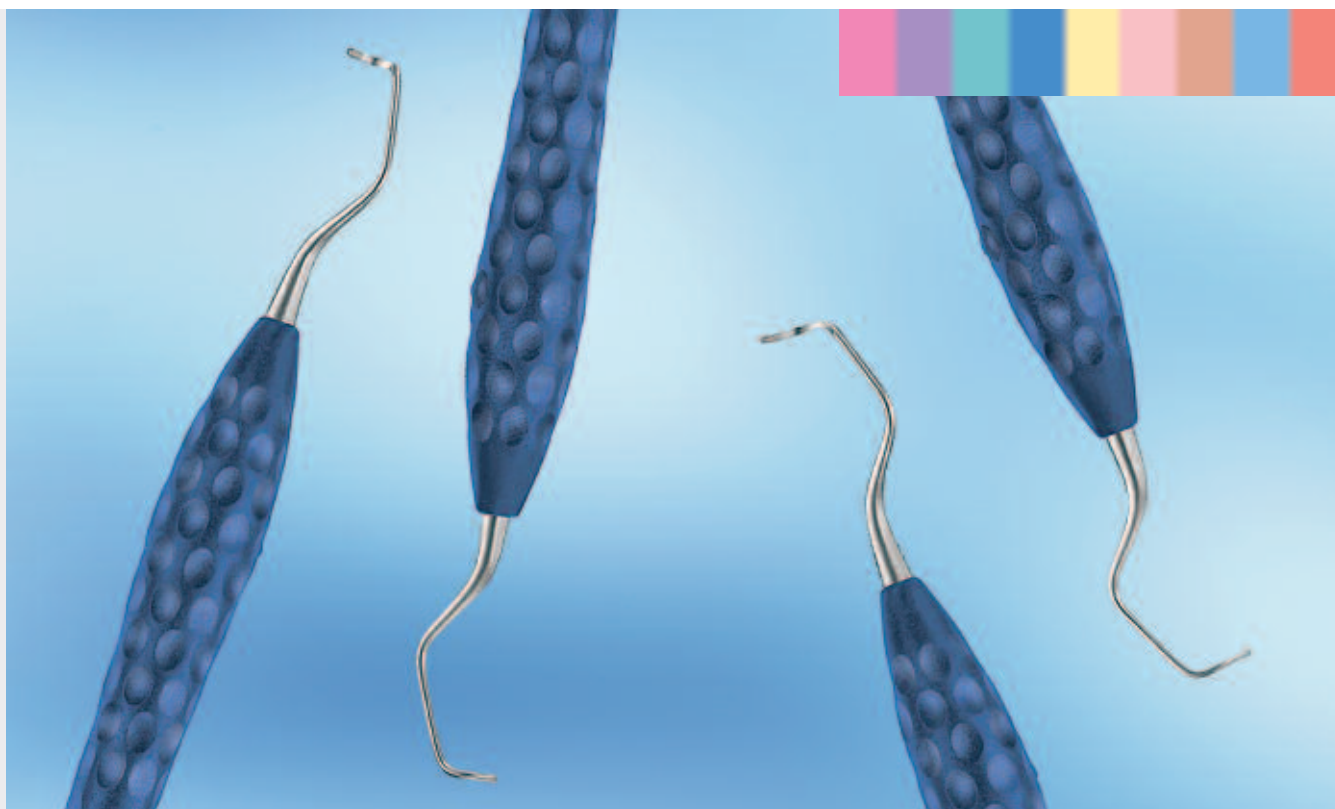
DX241R
M23A
ERGOPLANT
Curette universale
ERGOPLANT
Universal curette
190 mm, 7 1/2"

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To ensure a successful long-term implantation, a consequent post-operative treatment is very important. To eliminate plaque on the implant - abutment - interface without damaging the implant titanium curettes are the optimal choice.

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ERGOPLANT
Titanium universal curette, for im-
plants up to diam. 4.0 mm

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ERGOPLANT
Titanium universal curette, for im-
plants up to diam. 5.5 mm

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These instruments were developed to hold back lips, cheeks, tongue and lingual mucous membrane. Due to the ergonomic handle design, a more relaxed and fatigue-free working is possible, even in long operations.

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The mouth wedges are particularly advantageous for longer implantations and also provide support for the patient.

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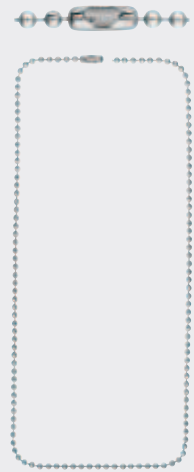
Silicone mouth wedges

Sales unit: PAK = Package of 2 pieces suitable for steam sterilization 134°C (Autoclav)

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Cuneo interdental in silicone con anima in metallo Taglia: M (media)

ERGOPLANT

Silicone mouth wedge with metal core
Size: M

1/1

DX115

ERGOPLANT

Cuneo interdental in silicone con anima in metallo Taglia: L (grande)

ERGOPLANT

Silicone mouth wedge with metal core
Size: L

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ERGOPANT
Handle for micro-
surgery blades

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Titanium scalpel handle for micro-
surgery blades, round handle. Light,
rotatable and handy for precise cutting

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Microsurgery blade
Sales unit: **PAK** =
Package of 10 pieces

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Microsurgery blade
Sales unit: **ST** =
Package of 1 piece

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The start of a new extraction culture.

With a new method to extract teeth in the shortest possible time without damaging the alveoli. The innovative AESCULAP exCenter extraction system offers safe and modern tooth extraction with many advantages for both user and patient.

Advantages over conventional methods:

- Almost no trauma for the patient
- Sensitive extraction with no force required
- No lever pressure on tooth
- Practically eliminates tooth crushing
- Alveoli remain to a large extent undamaged
- Preserves the buccal lamella
- Also ideal for immediate implantation
- Prevents inflammation and abscess formation
- Only one extraction forceps for all teeth
- Conventional tooth extraction forceps not required
- Ergonomic design
- Clearly organised instrumentation
- Safe preparation and storage in AESCULAP trays

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exCenter II extraction forceps without working tip, universal for upper- and lower jaw, function only with working tips DH822R – DH825R (to be ordered separately)

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exCenter II working tip 1
for upper incisors, upper and lower Canines, Premolars

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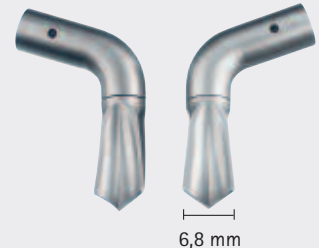
exCenter II working tip 2
for lower incisors, upper and lower roots

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exCenter II working tip 3
for upper and lower Molars, on intact crowns

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exCenter II working tip 4
"Bifurcation"
for upper and lower Molars, on partly damaged crowns

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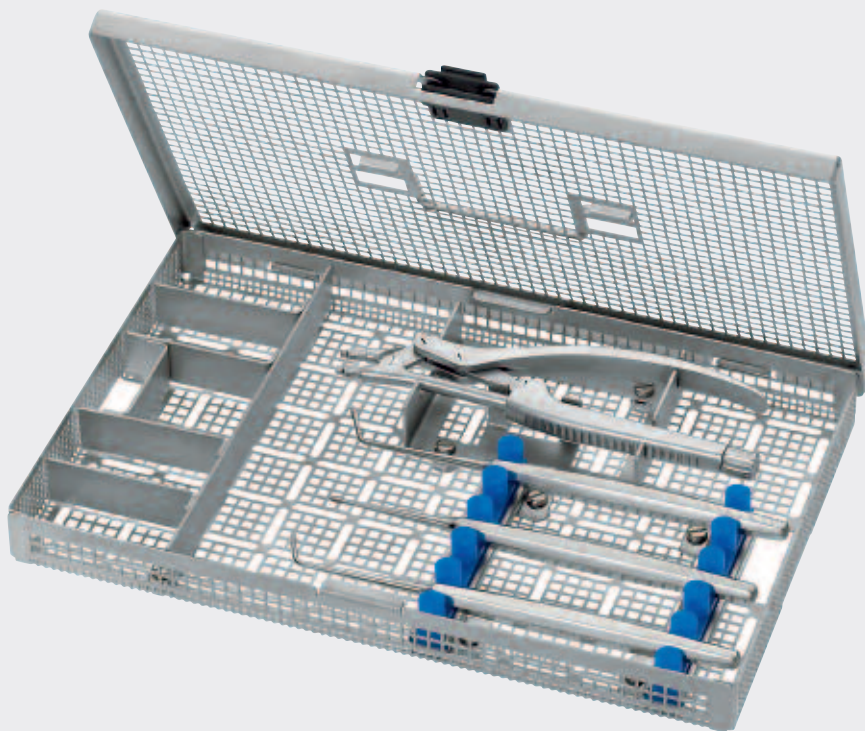
DH820

Aesculap exCenter II Extraction System, consisting of:

- DH821R 1 Eccentric II tooth forceps without tip
- D0801R 1 Desmotome, straight, filigree
- D0802R 1 Desmotome, left, filigree
- D0803R 1 Desmotome, right, filigree
- JF267R 1 Tray (274 x 172 x 30 mm)
suitable for dental container JN092
- TE884 1 Packing stencil (only available for DH820)

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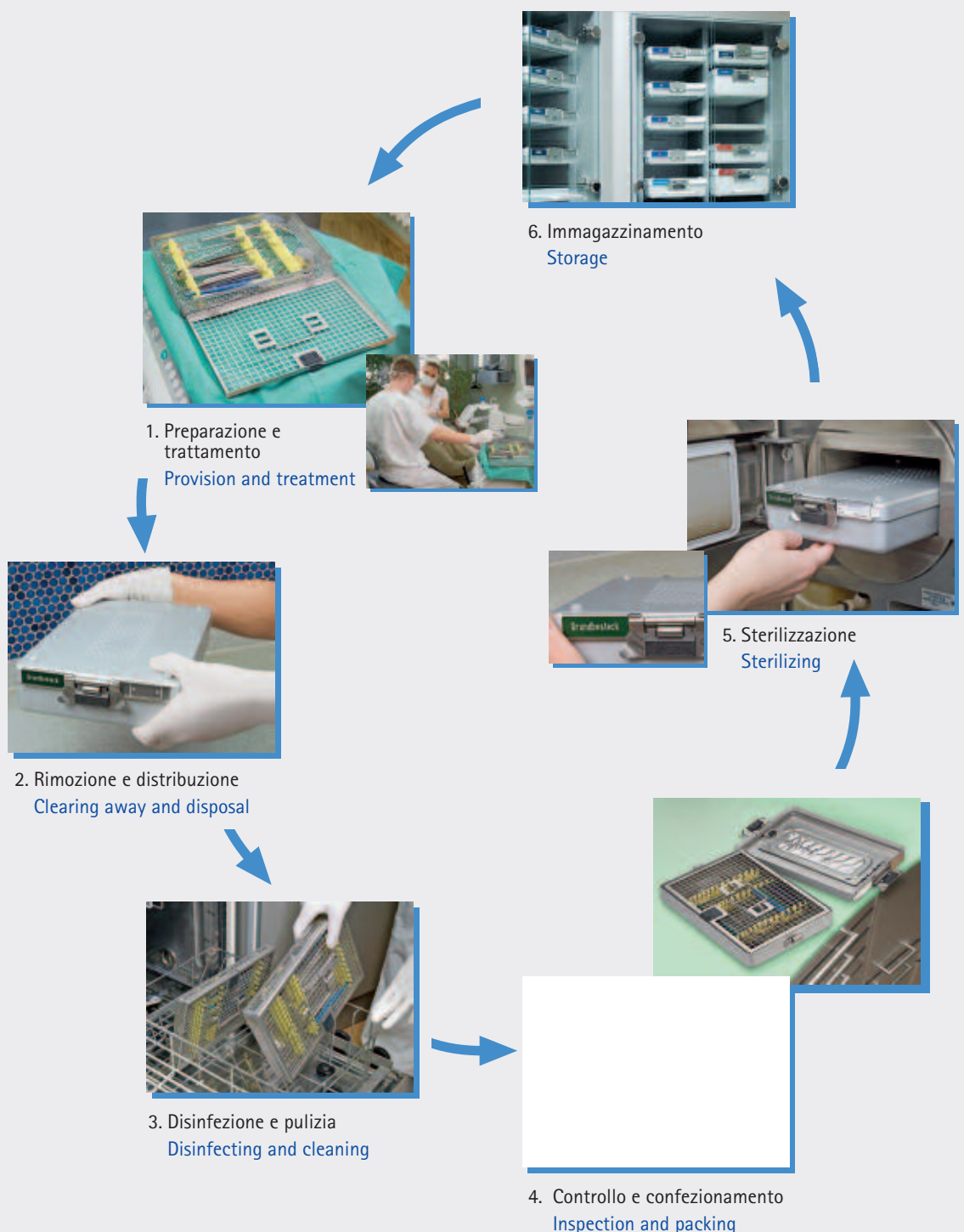
The sterile goods cycle in the dental practice with the Aesculap Sterile Container System:

All hygienic measures in the dental practice are primarily for keeping healthy the patients, the dentist and the assistants.

To ensure the best possible safety it is particularly important to observe the principles of hygiene when reprocessing instruments.

To keep practice work safe, efficient and economical, clear procedures must be specified for practice hygiene.

The Aesculap Sterile Container System has proven itself as an excellent aid for organizing the instrument reprocessing procedure.

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No. catalogo Cat.No.	à pagina Page	No. catalogo Cat.No.	à pagina Page	No. catalogo Cat.No.	à pagina Page	No. catalogo Cat.No.	à pagina Page
BB045T	9, 58	DO802R	64, 65	DX360T	53	DX573R	15, 38
BB063R	5, 9, 59	DO803R	64, 65	DX361T	53	DX574R	15, 38
BB363R	58	DO804R	64	DX400R	7, 50	DX575R	15, 39
BB364R	58	DO805R	64	DX401R	9, 42	DX576R	5, 39
BB367R	9, 58			DX402R	9, 42	DX577R	5, 39
BB369R	58	DX050R	9, 41	DX405R	9, 42	DX578R	5, 39
BB511	59	DX051R	5, 41	DX450R	9, 40	DX579R	5, 39
BB512	59	DX052R	5, 41	DX500R	5, 22	DX600R	7, 21
BB515	5, 9, 59	DX055R	9, 41	DX501R	7, 22	DX705	5
BB515-C	59	DX094	5, 40	DX505R	28, 29	DX706	7
BB542	59	DX100R	54	DX506T	29	DX707	9
		DX101R	55	DX507T	29	DX709	13
BC061R	9, 60	DX102R	55	DX509R	29	DX710	15
		DX105R	56	DX510R	11, 26	DX720	44
BH111R	60	DX106R	56	DX511R	11, 26	DX800	17
		DX110R	22	DX512R	11, 26	DX801R	17, 18, 19
BM003R	9, 44, 47, 61	DX113	57	DX513R	11, 27	DX802R	18, 19
BM065R	5, 61	DX114	57	DX517R	11, 27	DX803R	17, 18, 19
		DX115	57	DX518R	11, 27	DX807	11
DA036R	5, 40	DX120R	57	DX522R	11, 25	DX810R	17, 20
DA037R	40	DX200R	5, 43	DX530R	13, 30, 31	DX811R	17, 20
		DX201R	5, 43	DX531R	13, 30, 31		
DH820	65	DX202R	9, 48	DX532R	13, 30, 31	FD252R	44, 46
DH821R	63, 65	DX203R	9, 48	DX533R	13, 30, 31	FD253R	44, 46
DH822R	63	DX220R	5, 49	DX534R	13, 30, 31		
DH823R	63	DX230R	7, 49	DX535R	13, 32	GD580R	59
DH824R	63	DX231R	7, 49	DX536R	13, 32	GD584R	59
DH825R	63	DX232R	49	DX537R	13, 32	GD585R	59
		DX240R	50	DX538R	13, 32		
DO219R	5, 60	DX241R	7, 50	DX539R	13, 32	JF244R	5, 7, 9
DO571R	34	DX250R	51	DX545R	25	JF267R	65
DO572R	36	DX252R	7, 23, 51	DX550R	21	JF283R	11, 13, 15
DO573R	34	DX253R	7, 23, 51	DX555R	7, 21	JF284R	17
DO574R	35	DX254R	23	DX560R	33		
DO575R	35	DX300R	7, 51	DX561R	33	JG381R	44, 47
DO576R	36	DX301R	43, 44, 45	DX570R	15, 38		
DO577R	37	DX303R	44, 45	DX571R	15, 38	TE884	65
DO801R	64, 65	DX305R	44, 45, 58	DX572R	15, 38	TE888	17

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Aesculap AG